



For more detailed instructions on how to properly operate and use a sling with your lift, please refer to your lifts' operating manual.

WY' EAST Medicals policy is one of continuous development and improvement, and therefore, reserves the right to change these specifications without notice.

If you have any questions regarding these instructions or your lift, please contact WY' EAST Medical.

### To Reorder

Order a case of 5 slings.  
Ask for stock number 51170

Wy' East Medical Corporation  
P.O. Box 1625  
16700 S.E. 120<sup>th</sup> Ave.  
Clackamas, OR 97015, USA


(503) 657-3101  
(800) 255-3126  
FAX: (503) 657-6901

[www.wyeastmed.com](http://www.wyeastmed.com)  
[info@wyeastmed.com](mailto:info@wyeastmed.com)

51187-R1  
WY' EAST MEDICAL DISPOSABLE SLING OPERATING  
INSTRUCTIONS



## Patient Specific Disposable Sling

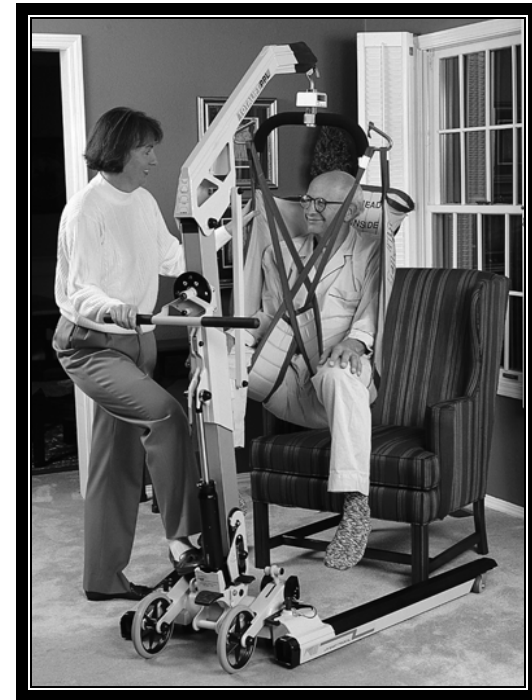

### Operating Instructions



51187-R1  
WY' EAST MEDICAL DISPOSABLE SLING OPERATING  
INSTRUCTIONS



Read carefully the safety instructions and warnings before attempting to use the WY'EAST Medical DISPOSABLE sling.

### SAFETY INSTRUCTIONS AND WARNINGS

- ❑ These instructions are to be used in conjunction with those issued with your vertical lift.
- ❑ This DISPOSABLE sling is to be used as a single patient use only sling. It may be held for use by the same patient until it becomes soiled, contaminated, damaged, or until the patient no longer requires it.
- ❑ The sling must not be used in a bathing or showering application.
- ❑ Dispose of used sling in accordance with your facility's policy.
- ❑ Inspect the sling carefully before each use (see figure 1). If fraying, seam separation, or other damage is visible or suspected, dispose of it immediately and replace it with a new DISPOSABLE sling.
- ❑ The WY'EAST Medical DISPOSABLE sling is of a unisize design to fit patients up to a maximum of 450 lbs (204 kg).
- ❑ Never exceed the weight limitation of your lift or sling, as serious harm to the patient or caregiver may result.

### SLING CARE



DO NOT WASH



DO NOT DRY



DO NOT DRY CLEAN



DO NOT IRON

- ❑ Ignore any reference in your vertical lift instructions for laundry and care of slings, when using the WY'EAST Medical DISPOSABLE sling.
- ❑ WY'EAST DISPOSABLE slings are designed for a limited period of use only and by their nature of design must be treated as a disposable product.

WY'EAST Medical DISPOSABLE slings have been designed too be used in a manner that is consistent with that of your vertical lifts' standard washable slings.

Before using the DISPOSABLE sling, familiarize yourself with the proper method of sling operation found in your lifts, operators' manual.

The patient to be transferred must be assessed by a qualified professional before attempting to transfer.

Always inspect the sling before every use for any wear or damage and replace as needed (see figure 1).

When issuing a DISPOSABLE sling to a patient, always write the patients' name using a permanent marker in the space provided on the label located on the back of the sling (see figure 2).

Place the sling under the patient by using a rolling technique, if patient is on the floor or in bed. If patient is sitting up in a chair, slide the sling down the patients back from behind the patient's head.

Bring leg bands under and up between the thighs.

Route leg bands through each other as shown in figure 3 and connect to opposite ends of lifting bar.

Connect loops at head end of sling to the same side of lifting bar as shown in figure 4.

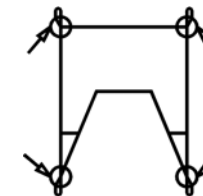


FIGURE 1  
SLING INSPECTION POINTS



FIGURE 2  
SLING LABEL



FIGURE 3  
CROSSED LEG BANDS



FIGURE 4  
LOOP CONNECTION