




Patient Specific Disposable Repositioning Sling



Operating Instructions

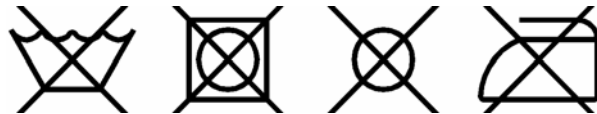

To Reorder

Order a case of 5 slings.
Ask for stock number 51411

Wy'East Medical Corporation
P.O. Box 1625
16700 S.E. 120th Ave.
Clackamas, OR 97015, USA

(503) 657-3101
(800) 255-3126
FAX: (503) 657-6901

www.wyeastmed.com
contact@wyeastmed.com

For more detailed instructions on how to properly operate and use a sling with your lift, please refer to your lifts' operating manual.

WY'EAST Medicals policy is one of continuous development and improvement, and therefore, reserves the right to change these specifications without notice.



Read carefully the safety instructions and warnings before attempting to use the WY'EAST Medical DISPOSABLE sling.

SAFETY INSTRUCTIONS AND WARNINGS

- These instructions are to be used in conjunction with those issued with your vertical lift.
- This DISPOSABLE sling is to be used as a single patient use only sling. It may be held for use by the same patient until it becomes soiled, contaminated, damaged, or until the patient no longer requires it.
- The sling must not be used in a bathing or showering application.
- Dispose of used sling in accordance with your facility's policy.
- Inspect the sling carefully before each use (see figure 1). If fraying, seam separation, or other damage is visible or suspected, dispose of it immediately and replace it with a new DISPOSABLE sling.
- The WY'EAST Medical DISPOSABLE sling is of a unisize design to fit patients up to a maximum of 600 lbs (273 kg).
- Never exceed the weight limitation of your lift or sling, as serious harm to the patient or caregiver may result.

- WY'EAST DISPOSABLE slings are designed for a limited period of use only and by their nature of design must be treated as a disposable product.

WY'EAST Medical DISPOSABLE slings have been designed too to be used in a manner that is consistent with that of your vertical lifts' standard washable slings.

Before using the DISPOSABLE sling, familiarize yourself with the proper method of sling operation.

The patient to be repositioned must be assessed by a qualified professional before attempting to reposition.

Always inspect the sling before every use for any wear or damage and replace as needed (see figure 1).

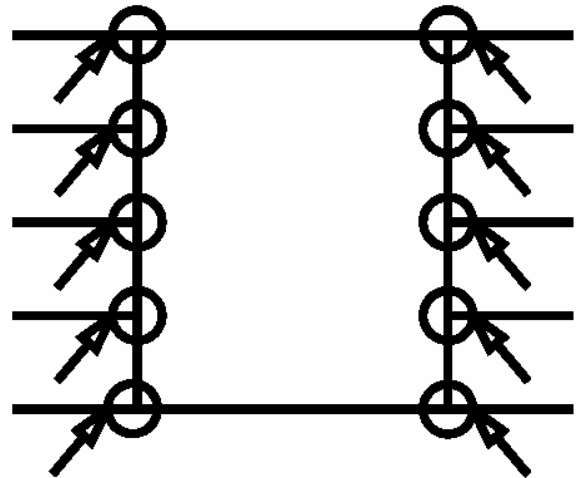


FIGURE 1
SLING INSPECTION POINTS

SLING CARE



- Ignore any reference in your vertical lift instructions for laundry and care of slings, when using the WY'EAST Medical DISPOSABLE sling.