

SlingSupply™

Sling Basics

SlingSupply * Clackamas. OR * 800-255-3126 * www.slingsupply.com

Re: SlingSupply Patient Slings, excluding disposable slings

Warranty:

All slings are warranted for 3 months from their delivery date. This warranty is not linked to the number of laundering cycles.

Construction:

Covering: 100% Dupont 1000d Cordura ® nylon with urethane coating.

Foam: closed cell cross-linked polyolefin.

Strapping: Polypropylene filament yarn containing organic lubricant.

Stitching thread: bonded polyester

Laundering:

Water temperature 140°F to 212°F


Mild detergent

Tumble dry at warm temperature

No Chlorine oxidizer

Inspections:

Slings should be inspected prior to each use for any signs of wear or damage at the inspection points shown below and replace as needed.

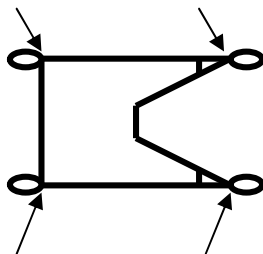

Instructions on Use:

- Place the sling under the patient by using a rolling technique, if patient is on the floor or bed
- If patient is sitting up in a chair, slide the sling down the patient's back form behind the patient's head
- Bring the leg bands under and up between the thighs
- Route leg bands through each other and connect to opposite ends of lifting bar
- Connect loops at head end of sling to the same side of lifting bar