

SlingSupply™

Sling Instructions

SlingSupply slings have been designed to be used in a manner that is consistent with that of your vertical lifts' standard washable slings.

Before using the sling, familiarize yourself with the proper method of sling operation found in your lifts, operators' manual.

The patient to be transferred must be assessed by a qualified professional before attempting to transfer.

Always inspect the sling before every use for any wear or damage and replace as needed.

Place the sling under the patient by using a rolling technique, if patient is on the floor or in bed. If patient is sitting up in a chair, slide the sling down the patients back from behind the patient's head.

Bring leg bands under and up between the thighs.

Route leg bands through each other as shown in **Figure 1** and connect to opposite ends of lifting bar.

Connect loops at head end of sling to the same side of lifting bar as shown in **Figure 2**.

FIGURE 1
CROSSED LEG BANDS



FIGURE 2
LOOP CONNECTION



For more detailed instructions on how to properly operate and use a sling with your lift, please refer to your lifts' operating manual.

SlingSupply's policy is one of continuous development and improvement and therefore, reserves the right to change these specifications without notice.

If you have any questions regarding these instructions, please contact SlingSupply.