





# Lateral Transfer QUICK INSTRUCTIONS


## Lateral Transfer without draw sheet

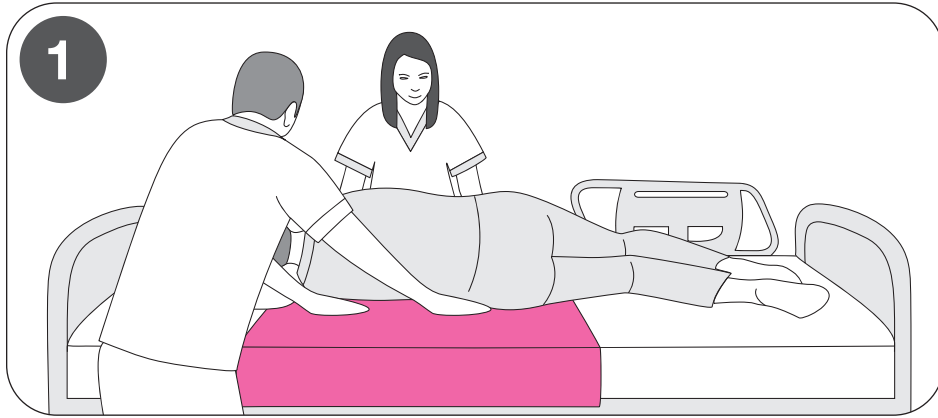

**1** **APPLY** the Pink Slip™ under the patient utilizing the “log roll” or similar technique.

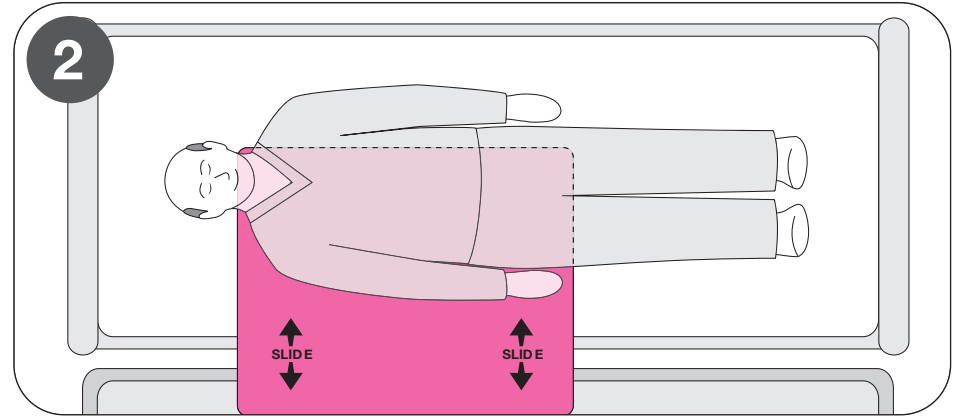

**2** **POSITION** the Pink Slip™ under patient so that its edge is parallel and just past the patient’s midline as shown.

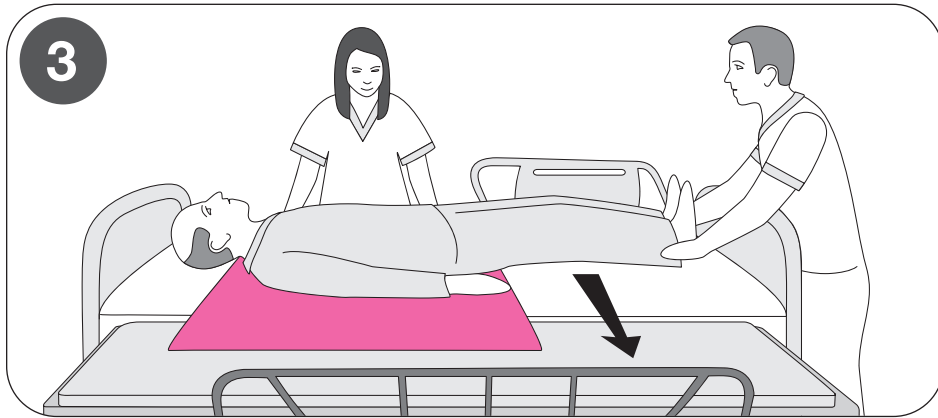


**3** **TRANSFER** patient by pushing on patient’s **shoulder** and just below **hip**, while the other caregiver lifts patient’s legs. This procedure should go easily and unforced.




**4** **REMOVE** the Pink Slip™ carefully by pulling on the bottom side of the Pink Slip™. Log roll patient if necessary to assist with this procedure.


## Lateral Transfer with draw sheet

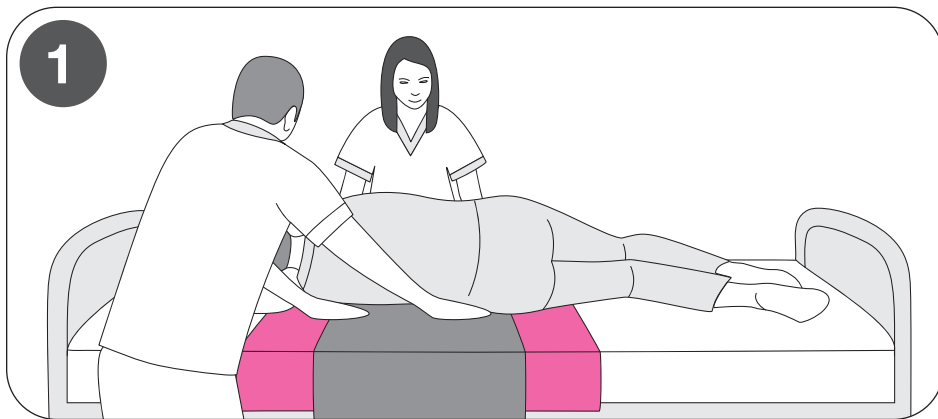

**1** **APPLY** the Pink Slip™ under draw sheet as shown utilizing the “log roll” or similar technique.

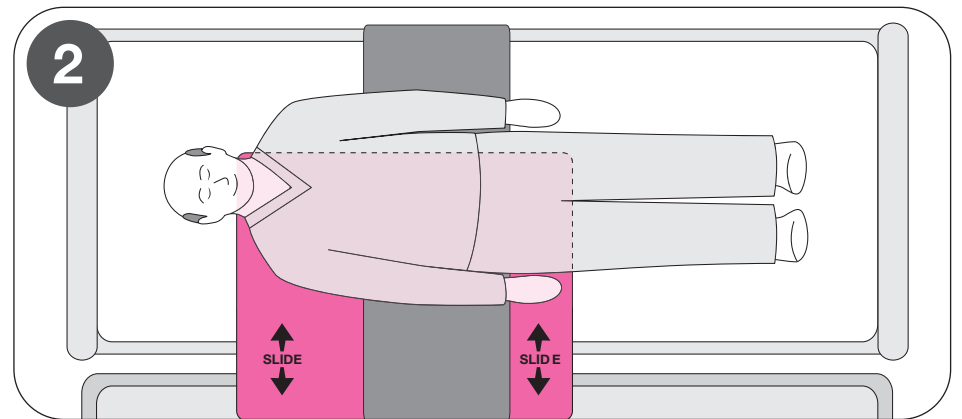

**2** **POSITION** the Pink Slip™ under patient so that its edge is parallel and just past the patient’s midline as shown.

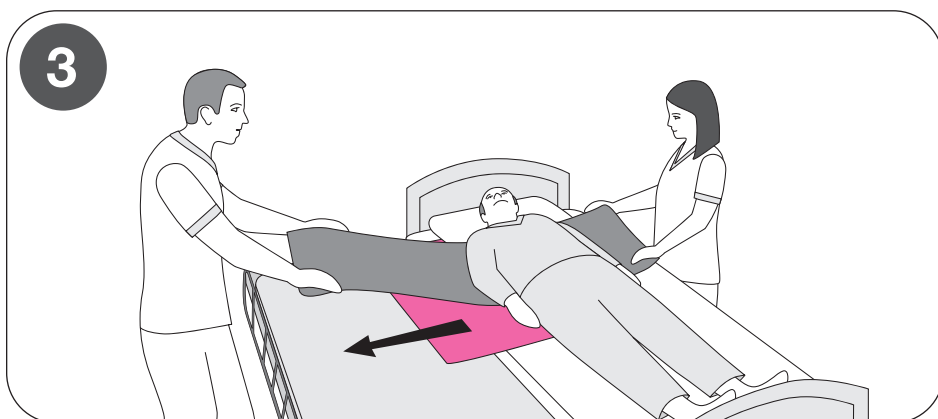

**3** **TRANSFER** patient with caregivers facing each other and patient between them. This procedure should go easily and unforced.



**4** **REMOVE** the Pink Slip™ carefully by pulling on the bottom side of the Pink Slip™. Log roll patient if necessary to assist with this procedure.

Please refer to detailed instructions and safety guidelines prior to use:

[WWW.PINKSLIPMED.COM](http://WWW.PINKSLIPMED.COM)




633045R00




# Reposition QUICK INSTRUCTIONS


## Reposition with one caregiver

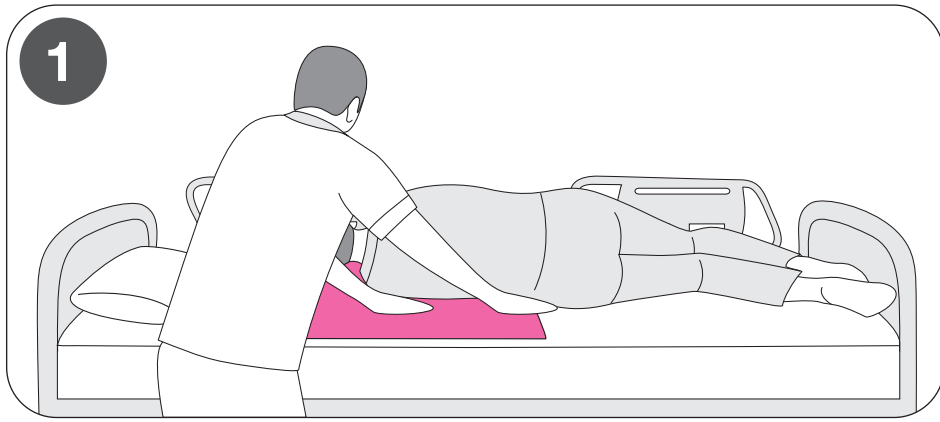


**1**  
**APPLY** the Pink Slip™ under the patient utilizing the “log roll” or similar technique.

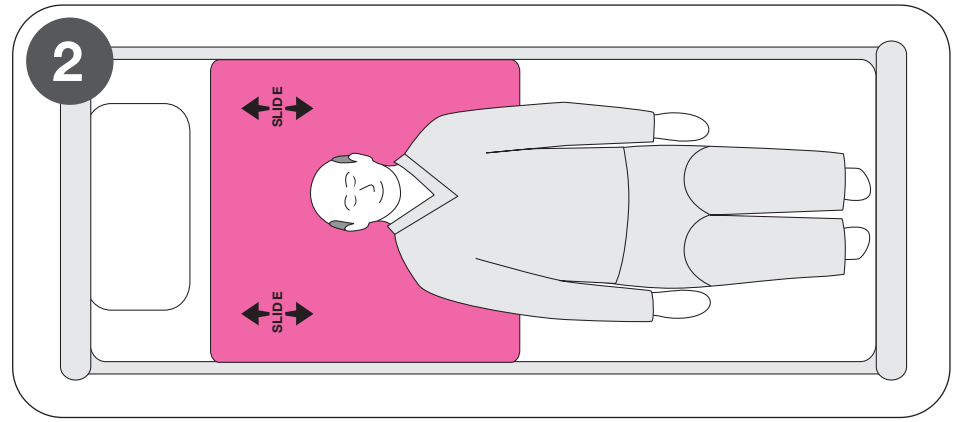

**2**  
**POSITION** the Pink Slip™ completely under patient and approximately 12 inches above patient’s head as shown.

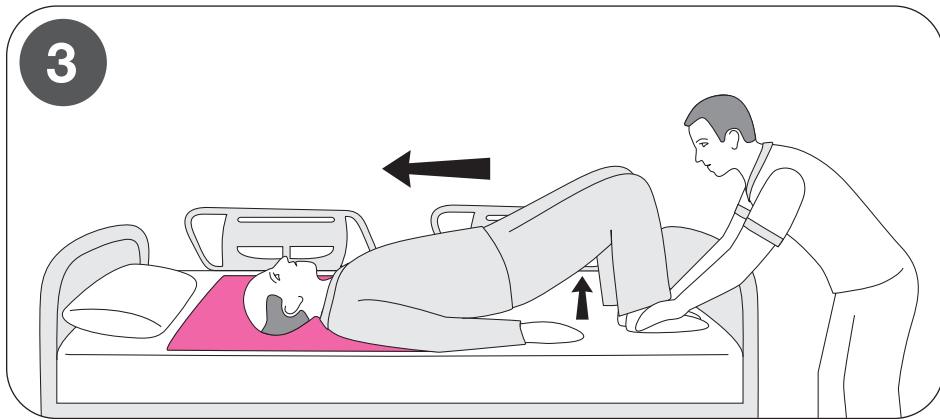


**3**  
**REPOSITION** by instructing patient to bend their knees and lift buttocks slightly. Apply pressure against patient’s feet to increase traction. Next, instruct patient to push themselves with their knees upward in bed.

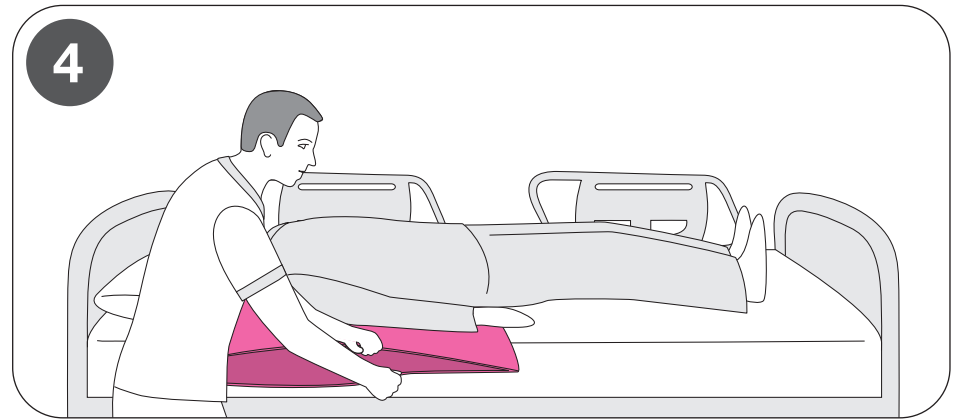


**4**  
**REMOVE** the Pink Slip™ carefully by pulling on the bottom side of the Pink Slip™. Log roll patient if necessary to assist with this procedure.


## Reposition with two caregivers

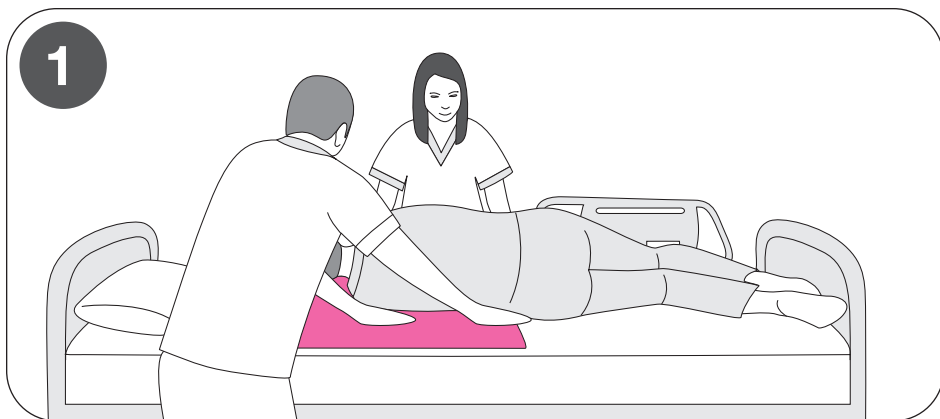


**1**  
**APPLY** the Pink Slip™ under the patient utilizing the “log roll” or similar technique.

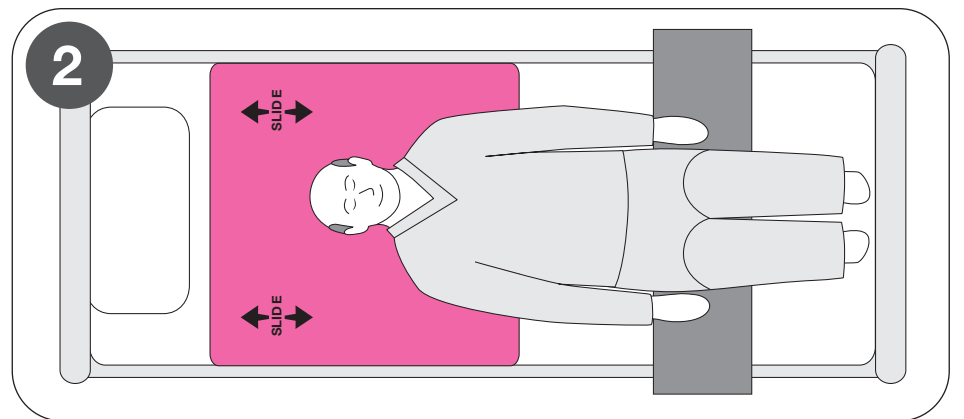

**2**  
**POSITION** the Pink Slip™ completely under patient and approximately 12 inches above patient’s head as shown. Apply draw sheet under patient’s legs then gently pull until it is next to patient’s buttocks.

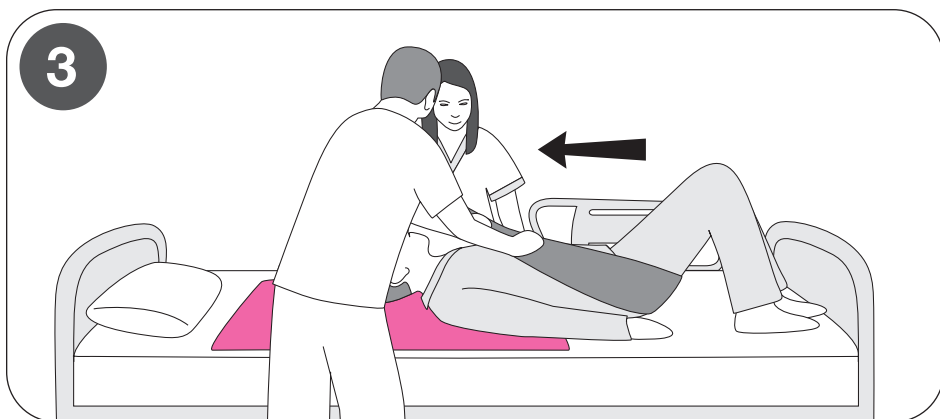

**3**  
**REPOSITION** patient by pulling on draw sheet as shown. Caregivers should be in line with the patient’s shoulders with a firm hand-hold of the draw sheet. The draw sheet should be approximately 20 degrees from horizontal.

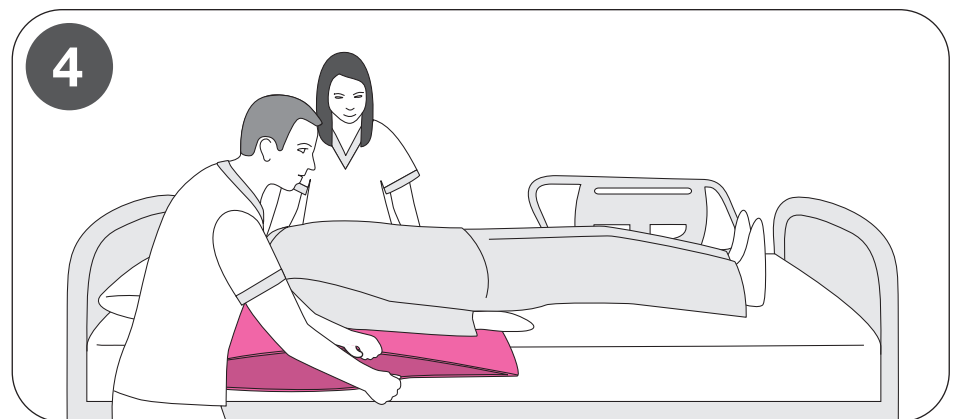

**4**  
**REMOVE** the Pink Slip™ carefully by pulling on the bottom side of the Pink Slip™. Log roll patient if necessary to assist with this procedure.

Please refer to detailed instructions and safety guidelines prior to use:

[WWW.PINKSLIPMED.COM](http://WWW.PINKSLIPMED.COM)



633045R00